



MC-NC2-DL-22-RW

**Nurse Call-Dome Light
2 Color, 2 Input – Red/White**

The **MC-NC2-DL-22-RW** is a stylish, exceptionally bright Nurse Call Dome Light. It is the central tie point for the Patient Room, with wiring going to the Call Stations and to the Central Control Module. The Call stations are equipped with Momentary Switches that are monitored and latched in the Central Control Module. The Call stations have Call Confirmation Indicators that display the same output as the Dome Light.

The **MC-NC2-DL-22** supports Normal Calls and Emergency Calls. When no calls are originating from a room, the Dome Light is Off. When a "Normal Call" is placed, the Dome Light illuminates the White Light. When an "Emergency Call" is pending, the Red Dome Light flashes. Calls can be cancelled from the Station where the Call was originated or by being answered by the operator. Call Cancellation is completely programable. The Dome Light will only turn off when calls are cancelled at all Call Stations.

The **MC-NC2-DL-22** are equipped with RJ45 CAT5 connector for installation convenience. A single Dome Light can monitor and display 2 system station ports for discrete audio to 2 patient stations in a single room.

The **MC-NC2-DL-22-RW-T** has an Optional Tone unit built in to make a local beeping sound when the station is active.

BENEFITS

- Reliable Signaling
- No Maintenance
- Low Power
- Simple Wiring
- Easily Integrates to other Systems
- Attractive Station Hardware

Associated Equipment

MC-IP-116-NC2	Audio Intercom Control Module
MC-NC2-VPBS	Bed Station
MC-NC2-VP-SAS	Staff Assist Station

Design Information

Power	24V DC @ 80mA
Color	White
Mounting	1 or 2 Gang Back Box
Dimensions	5" W x 3" D x 5" H
Weight	1 lb.




Tech Works®

"Making Specialized Communication Easy"



Quick Start Installation Instructions

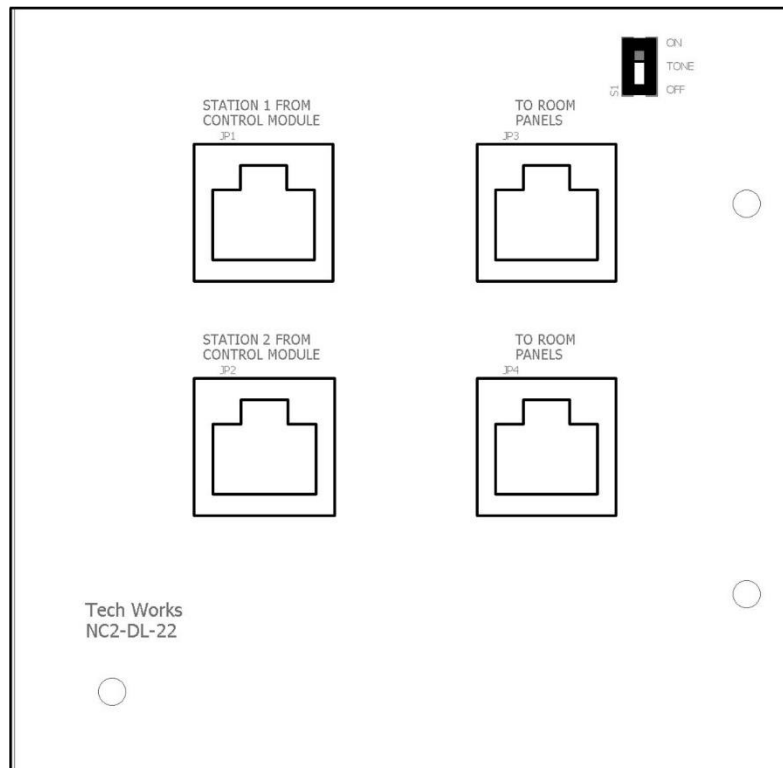

Wiring

The MC Stations have a variety of options and features that are unique to each product. All Station wiring is a CAT5 or CAT6 in and out. The installer then chooses if the call is and “E” for Emergency or an “N” for a normal call to set the station priority and the Channel which set the top or bottom of an associated dome light.

An integral Tone Option is available under the same part #, but with a “T” on the end, for local audible signaling. The Tone “ON/OFF” switch allows the installer to quickly activate the tone if included and desired.

Architects' and Engineers' Specifications

The Nurse Call System Dome lights shall be LED type with super bright light output and wide viewing angle. Dome Lights shall be supervised for proper Operation. Two call station inputs shall provide signaling of both Normal and Emergency Call from each room. An integral tone shall be available for local audible signaling. The Tone “ON/OFF” switch shall allow the installer to quickly activate the tone.

The Nurse Call System Dome Lights shall be: Tech Works Model MC-NC2-DL-22-RW

